




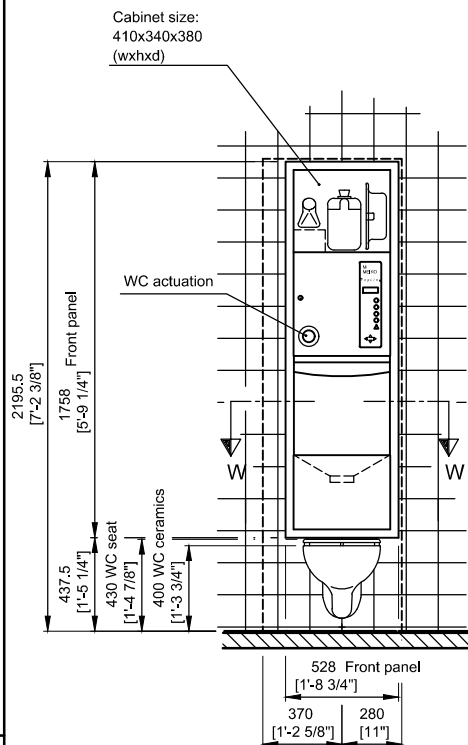


Partial delivery 1

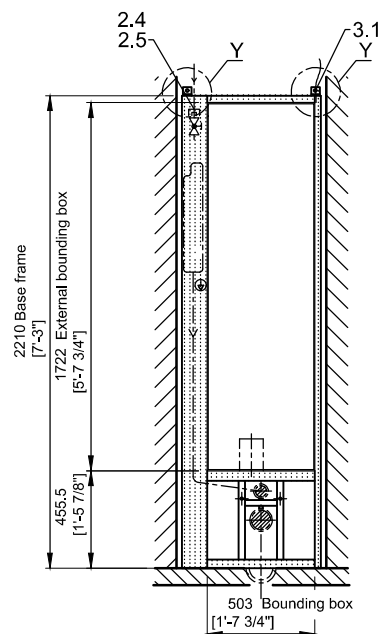


Partial delivery 2

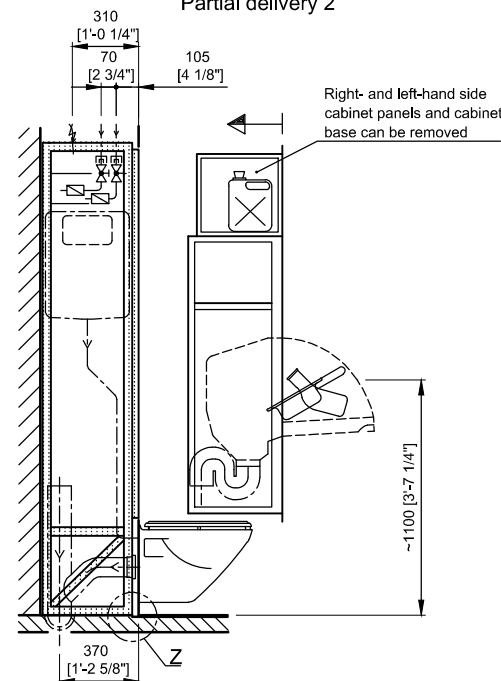




Detail Y

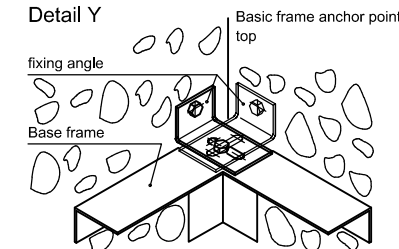

Detail Z

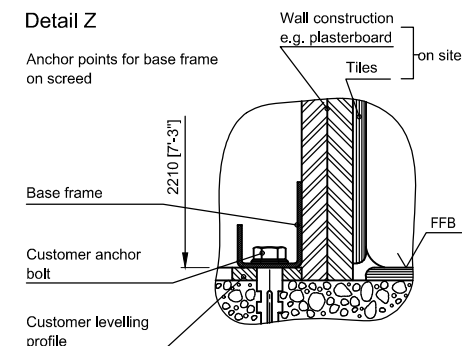

Section W-W

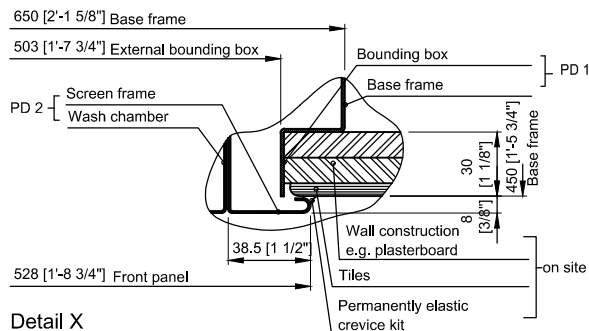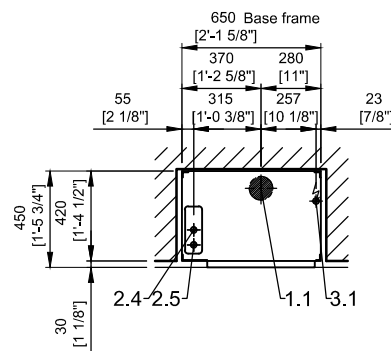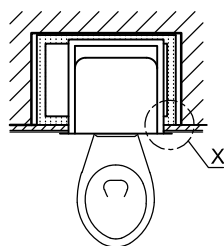

Detail X

Mounting base frame on onsite screed. WC ceramic height: 400 [1'-3 3/4"] mm +/- 10 [3/8"] mm (applies to all wall WCs in accordance with DIN EN 33:2011 with a distance between centres of bolts of 180 [7 1/8"] mm). When installing on raw floor or with a different ceramic height, an additional compensating frame can be supplied. (See detail Z).

Appliances intended to be installed in enclosed alcoves.

Observe protective measures when installing in a room with bathtub or shower

Sanitary installation		MEIKO	on site
1.1	Drain piping TopLine - WC	drain piping PE ø 110mm	DN 100 [4 "] floor sleeve DN 90 WC connection
2.4	Hot water	R 1/2", stop valves, dirt collector	inlet pipe DN 15, 45-60°C flow: 18l/min., flow pressure: 100-500 kPa
2.5	Cold water	R 1/2", stop valves, dirt collector	inlet pipe DN 15, 5-25°C flow: 18l/min., flow pressure: 100-500 kPa

Elektro-Installation

3 Gnd ~ 208/230V, 60Hz, 3,9/4,6kW
Equipotential bonding (thread bolt M8)
Protection class: Equipment Class 1,
Pollution Degree 2, Installation Category II.
Main switch, connection box

Cable from OK basic frame about 1,5m[4]. Protection: 3x15A. Protective measures and connection of the equipotential bonding must be carried out according to IEC 364-5-54 and conform to the local and national regulations. If the appliance is hard wired, a multiple-pole main switch must be installed. The main switch must be compliant with IEC 60947-1 and IEC 60947-3. Install the main switch in the vicinity of the appliance and in an area that is easily accessible. It must be clear that the main switch is linked to the appliance.

attachment "Important information" to be considered!

-	Original	21.11.2023	rm

MEIKO USA, INC. 1349 HEIL QUAKER BLVD, LA VERGNE, TN 37086 UNITED STATES OF AMERICA TEL: (615) 399-6600 FAX: (615) 399-6620 WEB: www.meiko.us © 2017 MEIKO USA, Inc. All rights reserved.		It is neither allowed to give this drawing without our permission to a third party for information or reproduction, nor may it be used for competition purposes. We reserve all rights!	
Reference:		Type:	
-		TOPLINE30/WC DT-ND/V THERMAL DISINFEKTION DECENTRALIZED	
Drawing-No.:		Project-No.:	Order-No. KG:
P00041483		-	USA
Scale:	drawn:	checked:	Customer checked:
1:25	21.11.23 rm	23.11.23 og-mt	
			EN DEU A3