




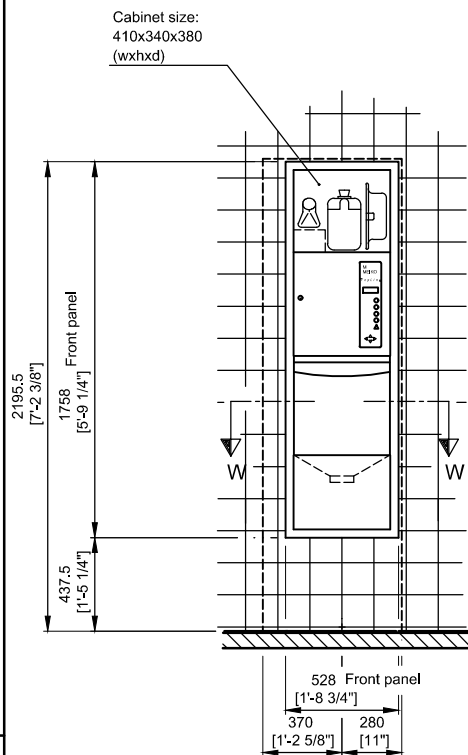


Partial delivery 1

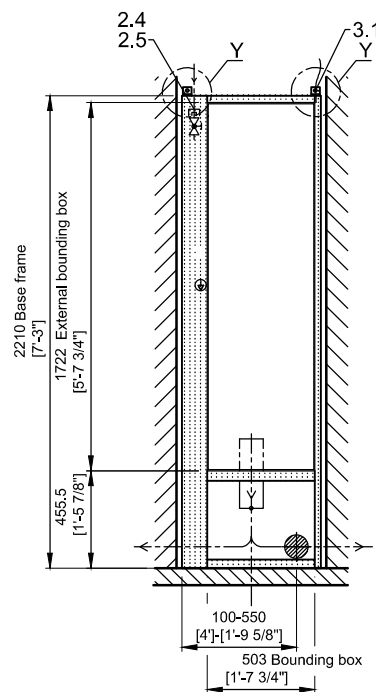


Partial delivery 2

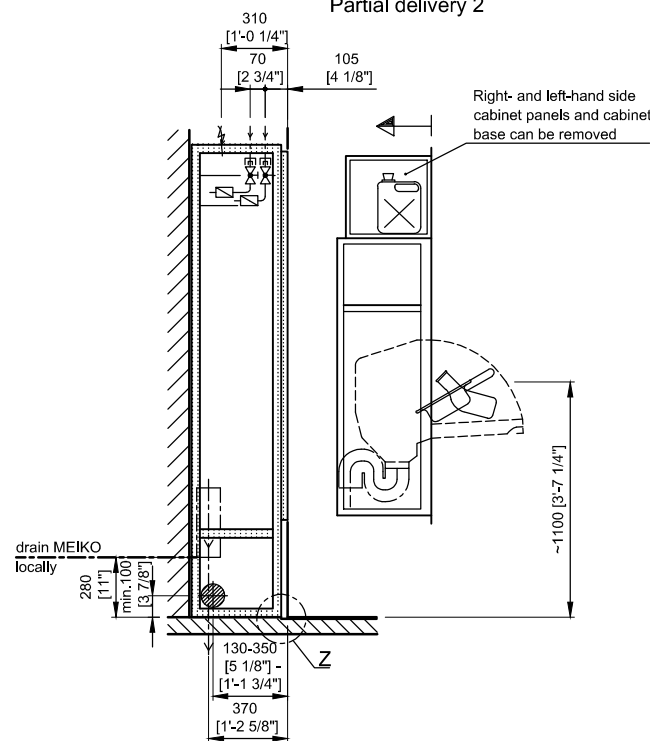




Detail Y

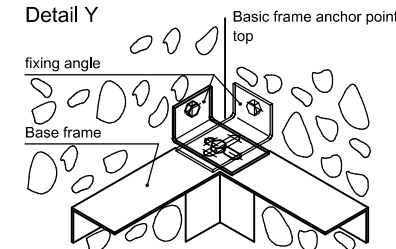

Detail Z

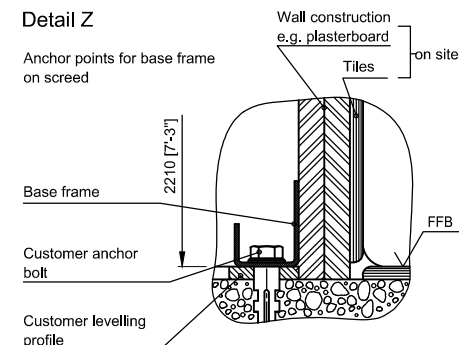

Section W-W

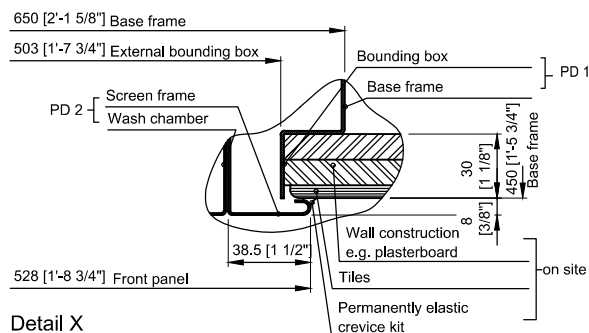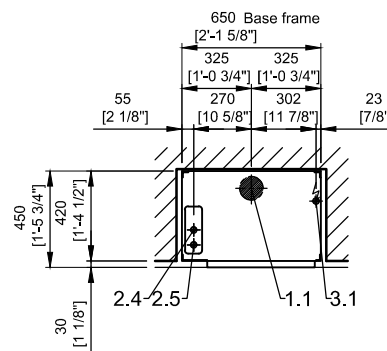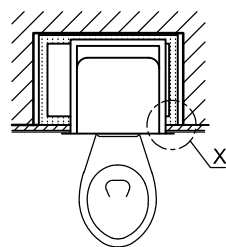

Detail X

Mounting base frame on onsite screed. (See detail Z).

Appliances intended to be installed in enclosed alcoves.

Observe protective measures when installing in a room with bathtub or shower

Sanitary installation		MEIKO	on site
1.1	Drain TopLine	Siphon PE ø 110mm	DN 100 [4"]
2.4	Hot water	R 1/2", stop valves, dirt collector	inlet pipe DN 15, 45-60°C flow: 18l/min., flow pressure: 100-500 kPa
2.5	Cold water	R 1/2", stop valves, dirt collector	inlet pipe DN 15, 5-25°C flow: 18l/min., flow pressure: 100-500 kPa
Elektro-Installation		Cable from OK basic frame about 1,5m[4"]. Protection: 3x15A. Protective measures and connection of the equipotential bonding must be carried out according to IEC 364-5-54 and conform to the local and national regulations. If the appliance is hard wired, a multiple-pole main switch must be installed. The main switch must be compliant with IEC 60947-1 and IEC 60947-3. Install the main switch in the vicinity of the appliance and in an area that is easily accessible. It must be clear that the main switch is linked to the appliance.	
3.1	3 Gnd ~ 208/230V, 60Hz, 3,9/4,6kW Equipotential bonding (thread bolt M8) Protection class: Equipment Class 1, Pollution Degree 2, Installation Category II. Main switch, connection box		

attachment "Important information" to be considered!

-	Original	21.11.2023	rm

 MEIKO USA, INC. 1349 HEIL QUAKER BLVD, LA VERGNE, TN 37086 UNITED STATES OF AMERICA TEL: (615) 399-6600 FAX: (615) 399-6620 WEB: www.meiko.us © 2017 MEIKO USA, Inc. All rights reserved.		It is neither allowed to give this drawing without our permission to a third party for information or reproduction, nor may it be used for competition purposes. We reserve all rights!	
Reference:		Type:	
-		TOPLINE30 DT-ND/V THERMAL DISINFEKTION DECENTRALIZED	
Drawing-No.:		Project-No.:	Order-No. KG:
P00041485		-	USA
Scale:	drawn:	checked:	Customer checked:
1:25	21.11.23 rm	23.11.23 og-mt	
			EN DEU A3